



Compass

MADE IN THE USA

PRODUCT # **3095**
HCPCS – L1832



BENEFITS:

- Medial and lateral range of motion hinges
- 12" knee orthosis
- Open patella for additional knee cap stability
- Popliteal cut out for enhanced comfort
- Velcro® straps for firm, adjustable compression

XS 12" – 13"*

S 13" – 14"*

M 14" – 15"*

L 15" – 17"*

XL 17" – 19"*

2X 19" – 21"*

3X 21" – 22"*

4X 22" – 23"*

*knee circumference

WHERE SUPERIORITY MEETS SERVICE.

For over 15 years, Banyan Healthcare has stayed true to its two overarching principles. One, develop quality, innovative orthopedic and comfort care products, and two, provide a high level of service to all of our customers and partners. Whether it's for our Compass or any other product in our range of offerings, we focus on going beyond the expected. Whatever your needs may be, we consistently rise to meet them.



1

Unfasten the Velcro® closures and straps to open the brace.

2

Remove the adjustable range of motion hinges from both sides.

3

Adjust the hinges to the desired point (the degrees are notated on the hinges).

4

Replace the hinges in the brace.

5

Wrap the brace around the leg and secure the Velcro® at the thigh and the shin. Make sure the patella is situated in the patella opening.

6

Insert the straps into the buckles and secure to the desired compression.



Care Instructions:

Remove the hinge from the brace. Hand wash with mild soap in cold water. Air dry flat. Do not use hot water or hot dryer. Do not bleach.

MADE IN THE USA