



# Eliminator

MADE IN THE USA

PRODUCT #

**3077**

HCPCS - L1843



## BENEFITS:

- Revolutionary, patent pending polycentric hinge
- Can be used as either a medial or lateral unloader
- Universal design allows for either the right knee or the left knee
- Provides superior unloading capabilities
- Varus and valgus adjustable
- Adjustable flexion and extension joint
- Ideal for patients with unicompartmental osteoarthritis
- Foam condyle pad provides increased comfort
- Posterior thigh and calf straps for added suspension and rotation control
- Open brace design allows for ease of applying

## WHERE SUPERIORITY MEETS SERVICE.

For over 15 years, Banyan Healthcare has stayed true to its two overarching principles. One, develop quality, innovative orthopedic and comfort care products, and two, provide a high level of service to all of our customers and partners. Whether it's for our Eliminator or any other product in our range of offerings, we focus on going beyond the expected. Whatever your needs may be, we consistently rise to meet them.

**XS** 12" – 13"\*

**S** 13" – 14"\*

**M** 14" – 15"\*

**L** 15" – 17"\*

**XL** 17" – 19"\*

**2X** 19" – 21"\*

**3X** 21" – 22"\*

**4X** 22" – 23"\*

\*knee circumference



- 1 Insert the hinge into the pockets on the desired side of the brace.
- 2 Place the condyle pad under the hinge condyle.
- 3 Open all closures and straps.
- 4 Using the finger loops pull the brace up the leg until the patella is situated in the patella opening.
- 5 Close and adjust the thigh fasteners in the front of the brace.
- 6 Fasten and tighten both closures at the back of the leg.
- 7 Adjust the flexion and extension setting to the desired degree on the condyle of the hinge.
- 8 Adjust the upper and lower unloader dial (turn to the right to increase unloading). Hold & set the degree of unloading by tightening the set nut located on the back side of the hinge (tighten firmly against the long hinge leg).
- 9 Thread the 2" straps through the buckle and adjust.
- 10 Thread the inner 1.5" straps through the opposing buckle and adjust.
- 11 Adjust all four straps so the hinge makes constant contact with the knee and provides the desired degree of unloading.



### Care Instructions:

Remove the hinge from the brace. Hand wash with mild soap in cold water. Air dry flat. Do not use hot water or hot dryer. Do not bleach.

**MADE IN THE USA**